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aHc-Hydrogel⁹⁰

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Instruction for Use

aHc-Hydrogel⁹⁰

Ready-to-use hydrogel for moistening of dry wounds, as well as for dissolving fibrin deposits and necrotic tissue.

Product properties:

aHc-Hydrogel⁹⁰ can be quickly and easily applied and remains as hydrogel on the wound surface. It keeps the wound moist, creates a protective film and thus promotes wound healing.

The application of aHc-Hydrogel⁹⁰ on the wound is non-irritating and pain-free, with no restrictions to duration of product use.

aHc-Hydrogel⁹⁰ is for long-term moistening. It keeps wounds moist and forms an ideal wound healing environment, supporting the natural wound healing process.

The aHc-Hydrogel⁹⁰ included in the

Preservatives (sodium hypochlorite) can also provide unspecific decontamination even in wounds.

Wounds odor can be quickly eliminated by the use of aHc-Hydrogel⁹⁰.

Application:

Shake the bottle well before use!

Before use and during change of the wound dressing and / or a wound debridement is recommended to remove exudate, fibrinoid deposits and other impurities.

After the wound cleansing spray aHc-Hydrogel⁹⁰ from a short distance as often until the wound is completely covered according to its condition. aHc-Hydrogel⁹⁰ can be used in combination with compresses, plasters and other dressing material.

There are no restrictions regarding the frequency and duration of use.

The preservatives in aHc-Hydrogel⁹⁰ can cause a slight smell of chlorine.

Application restrictions:

There are no known side effects. aHc-Hydrogel⁹⁰ is contraindicated in patients with known sensitivity or allergic reactions to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Interactions with other substances are not expected. During pregnancy and whilst breastfeeding aHc-Hydrogel⁹⁰ should be used with caution. Do not use aHc-Hydrogel⁹⁰ on bleeding or exuding wounds.

General safety instructions:

Only apply to external wounds. Do not swallow, avoid eye contact. Keep away from children. Only use intact, undamaged bottles.

Only use the products prior to the expiration date indicated on the bottle. Once opened, product can be used for a maximum of 90 days.

Ingredients:

Water, sodium hypochlorite <0.06% (electrochemically activated mineral saline solution), lithium magnesium sodium silicate

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Prompted GmbH